INSTRUCTIONS FOR USE OF AB DENTAL GERMANY UG (Haftungsbeschränkt) SURGICAL KIT

General precautions

Before using Surgical tools, the operating surgeon/practitioner in charge should carefully study the indications, contraindications, recommendations, warnings and instructions, as well as all other product specific information (technical product description, description of the surgical and restorative technique, catalogue sheet, etc.) and fully comply with them. The manufacture, the importer and the suppliers of Surgical tools products are not liable for complications, other negative effects or damages that might occur for reasons such as incorrect indications or surgical technique, unsuitable choice of material or handling thereof, unsuitable use or handling of the instruments, asepsis and so on. The operating surgeon is responsible for any such complications or other consequences.

Cautions and Warnings

- Implant Drills are to be used only by a licensed dentist who, has the specialization, skills and the appropriate training in dental implants and restoration, to assure successful treatment.
- Always wear Gloves when handling contaminated instruments.
- Eye protection must be worn to protect against ejected particles.
- Surgical mask must be worn to avoid inhalation of aerosol or dust generated.
- Inspect the drill for damage or wear before each use and discard/return to manufacturer defective drills.
- Ensure that the drill is fully seated and gripped in the hand piece collet before use.
- Maintain hand piece in good working order and correctly lubricated.
- Do not exceed the maximum speeds indicated in this leaflet.
- Avoid excessive drilling speed and/or drilling duration to avoid overheating and associated complications.
- Move the drill continuously when in use to avoid localized heating.

- Clean and sterilize the drills in accordance with the directions below before first use and before each reuse.
- We advise to replace the drill every 30 implants.

Introduction – Intended Use

Implant Drill is for preparation of the jawbone for dental implant. The Implant Drill Accessories assist the Implant Drill in its use.

The drills and accessories shank meets the requirements of ISO 1797-1.

General Information

All instruments must be cleaned and sterilized prior to use. The drills have been marked with sizes and color coded for ease of identification. Size marking and color coding should be used to select proper drill for each surgical procedure.

Implant Drills are susceptible to damage and wear and should be inspected before use.

The number of uses per drill will vary and depends on a variety of factors including bone density encountered, proper handling and cleaning. Over time, repeat sterilizations may affect cutting efficiency and color appearance. Cutting edges should present a continuous edge and appear sharp. Check the latch lock shank for wear to ensure the connection is not damaged. If inspection reveals sign of wear, damage, or unrecognizable color identification, replace the drill accordingly. We advise 6 – 10 uses at the maximum.

Cleaning and sterilization

Correct and careful maintenance of Implant Drills is extremely important. The recommended stages are as follow:

1. Before use, soak the drills in a mild, pH-neutral (Action 201or equivalent) detergent for 2.5 minutes or until cleaning can be performed. Avoid contact with phenol alcohol, chlorine, acid or quaternary ammonia.
2. Rinse the drills under a hard stream

- of water. Avoid water with high concentration of chlorine.
3. Flush the drill lumen with a hypodermic needle.
4. Use a nylon brush to rid the drill of additional debris caught in the irrigation channel.
5. Flush the drill lumen with a hypodermic needle.
6. Ultrasonic cleaning per the unit manufacturer's instructions. Use a low-sudsing, pH-neutral detergent. Do not allow the drills to contact any other metals.
7. Wrap the drills for sterilization with wraps washed in pH-neutral detergents or disinfectants.
8. Sterilize the drills by steam per the autoclave's instructions. 134°C/275°F for up to a maximum 6 minutes followed by a 30 minutes' dry cycle. Distilled water should be used to avoid surface stains. Make sure before use that the elements, inside the autoclave, are not rusted. Chemclave is NOT recommended.

The drilling technique

All drilling procedures should be performed at maximum 800 – 1200 RPM with copious irrigation. The use of sharp drills, sufficient irrigation, an in-and-out drilling motion, short cutting cycles, waiting for the bone to cool, and use of pilot drills in successively increasing sizes are essential.

Storage

The user must avoid all effects that could affect the product marking or shelf-life of the drills, the drill surface or the drill geometry such as unnecessary commotion, strains, heat, UV radiation, moisture, etc.

Drilling Procedure

ALL IMPLANTS | After good surgical exposure of the bony surface, the position for the implant should be determined and a guide hole should be made using our round-head burr, taken down into the cortical bone to the level of the neck beneath the burr

head. Do not attempt to drill deeper with the round burr. Using the guide hole for position, the color-coded drill bits will be utilized to drill the hole to the desired depth. The color coding on the bits indicates the diameter of the bit. The bits are used in graduated order to slowly increase the diameter of the implant hole until the desired diameter is reached. This will allow safe progression and decrease trauma to the surrounding bony structures. The accurate depth of the hole is determined by the length of each particular implant and is indicated by the depth lines around each bit, in order to allow good position of the implant in the bone so that its end is flush with the alveolar ridge.

Proper Care

By following these guidelines, you will protect your container and ensure many years of productive and satisfactory performance.

DMS 12758 E

Rev. 1120

AB Dental Germany UG (Haftungsbeschränkt)
Friedrichstrasse 95 10117 Berlin, Germany
Phone +49-30-609857880
Fax +49-30-609857889
www.ab-dentalgermany.com

CE 2797

	Symbol for Use-by date
	Symbol for Do not reuse ISO 15223-1:2012 (5.4.2)
	Symbol for Caution Indicated the need to consult the instructions for use for important cautionary
	CE marking of conformity Complies with the requirements of directive 93/42/EEC for medical device
	Symbol for Batch code Lot number
	Consult instructions for use
	Date of manufacture
	Manufacturer
	Has not been subjected to a sterilization process
	Federal law restricts this device to sale by or on the order of a Dentist

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